Invacare® Omega

H407

en	Stationary Toilet Chair User Manual	2
fr	Chaise-toilettes posée fixe Manuel d'utilisation.....	4
es	Silla inodora Manual del usuario	7
pt	Cadeira sanitária estacionária Manual de utilização.....	10

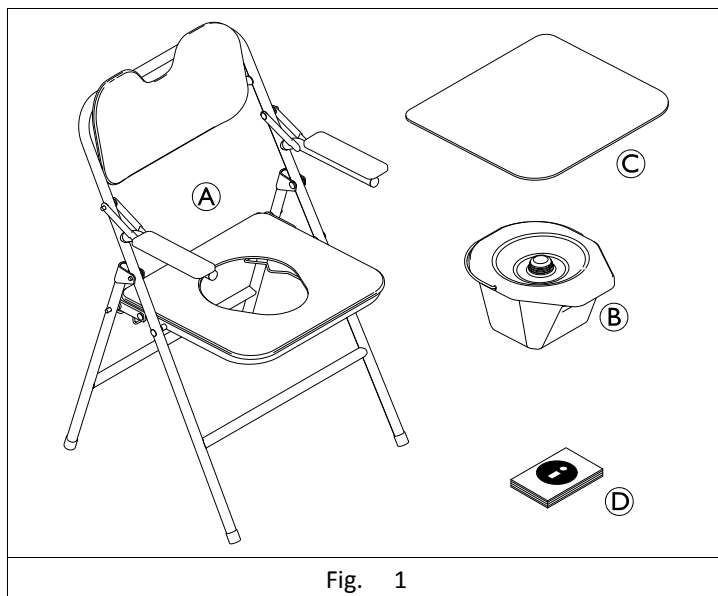
Fig. 1

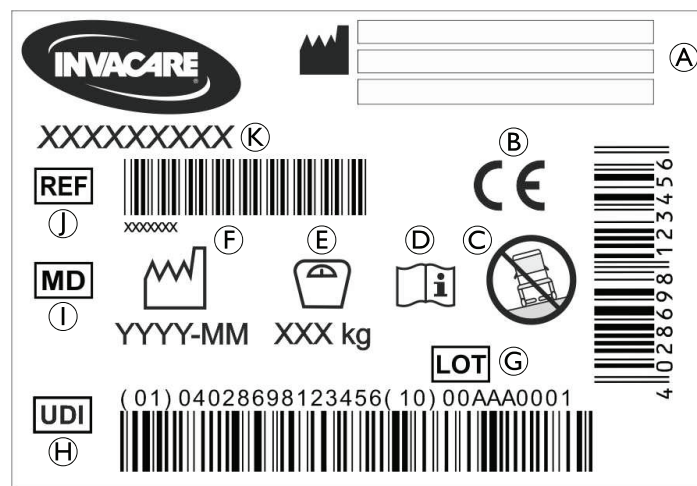
Fig. 2

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®

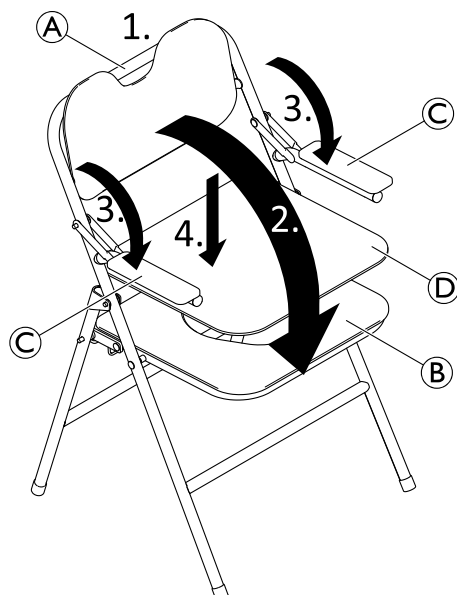
Fig. 3

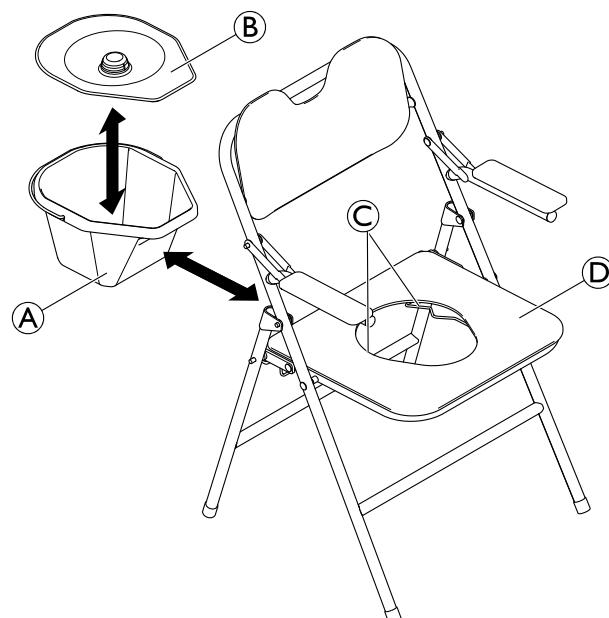
Fig. 4

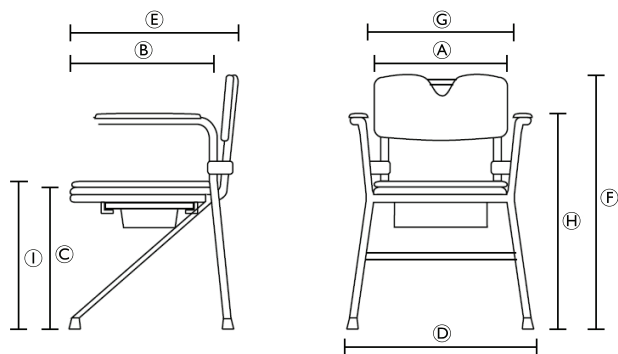
Fig. 5

Fig. 6

1 General

en

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

Product-specific Standards

The product has been tested and conforms to the standard ISO 17966 (Assistive products for personal hygiene that support users) and related standards. For further information about local standards and regulations, contact your local Invacare representative.

1.4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use

- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Injury!

Improper use of this product may cause injury.

- Before every use, check the product for damage and ensure that it is securely assembled.
- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not use or store the product outdoors.
- Do not use in the shower (Fig. 6).
- Keep product away from open flames.
- Do not place smoldering cigarettes on the product.

WARNING!

Risk of burning your Skin!

The product components can heat up when exposed to external sources of heat.

- Do not expose the product to strong sunlight before use.

WARNING!

Risk of Falling!

Improper use of this product may cause injury.

- Do not stand on this product.
- Do not move or displace the product under load or during use.
- Only use the product on level ground.

CAUTION!

Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor.

2.2 Labels and Symbols on the Product (Fig. 2)

Ⓐ	Manufacturer	Ⓔ	Lot number
Ⓑ	European Conformity	Ⓕ	Unique Device Identifier
Ⓒ	For use on level ground only	Ⓖ	Medical Device
Ⓓ	Read user manual	Ⓖ	Reference number
Ⓔ	Maximum user weight	Ⓚ	Product designation
Ⓕ	Production date		

The identification label is located under the seat on the tube.

3 Product Overview

3.1 Intended Use

The Invacare Omega H407 is a foldable toilet chair composed of padded armrest, backrest and a seat plate with toilet opening and a seat cover. A pan with lid can be placed under the seat. This product is intended to be used as a toilet aid providing stable seating position while going to toilet.

Intended User

Users are any adults and adolescents with restricted or limited mobility or limited muscle strength. If used without an assistant, the users must be able to keep their own balance.

Indication

- Restricted or limited mobility (e.g.: legs and/or hips) or
- having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended. For the maximum user weight, see 7 *Technical Data*, page 4.

3.2 Scope of Delivery

The scope of delivery is shown in figure 1.

4 Setup

Folding/Unfolding the Toilet Chair (Fig. 3)

CAUTION!

Risk of Pinching

- When folding or unfolding the toilet chair and the armrests, keep your hands away from the movable joints.

1. Hold the toilet chair with one hand at the backrest Ⓐ.
2. Fold down the seat Ⓑ with the other hand.
3. Fold down the armrests Ⓒ on both sides.
4. If desired, insert the seat plate Ⓓ from above to the seat, making sure that both stoppers on the underside engage into the opening for the toilet pan.
5. To fold, repeat steps 1 to 4 in reverse order.

Inserting/Removing the Toilet Pan (Fig. 4)

The toilet pan can be inserted/removed with or without the lid Ⓑ.

1. Insert the upper rim of the toilet pan Ⓐ from the rear side with the small side facing forward between the seat Ⓓ and the brackets Ⓒ under the seat.

5 Maintenance

5.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

5.2 Cleaning and Disinfection

General Safety Information

CAUTION!

Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

NOTICE!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

Cleaning Intervals

NOTICE!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product:

- before first use,
- regularly while it is in use,
- when it has been in contact with any body fluids.

Cleaning Instructions

NOTICE!

Maximum cleaning temperature:

- 60 °C by hand

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

6 After Use

6.1 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

6.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *5 Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

7 Technical Data

7.1 Dimensions, Weights and Materials (Fig. 5)

Dimensions			
Ⓐ	440 mm	Ⓕ	910 mm
Ⓑ	420 mm	Ⓖ	460 mm
Ⓒ	450 mm	Ⓖ	660 mm
Ⓓ	580 mm	Ⓖ	475 mm
Ⓔ	560 mm		

Weights	
Product weight	8.5 kg
Maximum user weight	130 kg

Materials	
Frame	Steel, powder coated
Upholstery	PVC
Toilet pan	PP

7.2 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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