**drive**

**DeVilbiss®**  
HEALTHCARE

**Kelis**

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## FOREWORD

We congratulate on the buying of a technically matured and the best available technology product.

This instruction helps you and the caregivers as a basis for sure, safe use of the toilet wheel chair and a sensitive use of the product's possibilities.

The instructions describe the preparation, service, use, servicing and cleaning of the product.

This instruction is to be perused carefully and must be always accessible to all users of the product.

## INDICATION

Toilet wheel chairs are necessary when using or achievement of the toilet is not possible because of a handicap. Toilet wheel chairs are also usable as room wheel chairs. Toilet wheel chairs can be pushed over customary toilet bowls, up to a height of 41 cm.

## SCOPE OF DELIVERY CONTENTS

1. Wheeled Drop-Arm Commode
2. Bucket with cover
3. Footrest
4. Heel tape
5. Operating instructions
6. 1 box containing casters and mounting screws.

## SAFETY INSTRUCTIONS

- Read this instructions carefully.
- Before use all components are to be checked for damages and firm seat.
- While seating, rising and while inserting the toilet bucket as well as folding up or engaging the armrests it's possible to pinch clothing or body parts.
- The use is allowed only on level grounds.
- Avoid moisture – Slide danger!
- With use of the toilet wheel chair always operate the full stop brake.
- Before getting in/from the toilet wheel chair swing away the footrests.

- Footrests never use as stand up device - Tilting danger!
- The steadiness of the toilet wheel chair is clearly lower as with wheel chairs, therefore avoid lateral lean out – Tilting danger!

- Protect from unauthorized use, e.g., by children, by locking the brakes.

- The toilet wheel chair serves excluding the use / transportation of a person to max. 130 kg on the seat and not as a transport possibility of objects (e.g., to ladders, buckets or similar)!

- **ATTENTION:** While driving reverse across door sills with the loaded commode chair it causes the risk that the interlock of the armrests get opened.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## REAPPLICATION

This product is suitable for reapplication.

With the processing the standards of the manufacturer must be considered.

## ASSEMBLY

The Wheeled Drop-Arm Commode is mountable nearly without tools.

Put the pre mounted backrest on the basic rack and protect with the star screws with a clockwise rotation.

Install the wheels using the screws provided.


## INTER- AND UNLOCK OF ARMRESTS

Pull both armrests upwards until they audible engage the interlock. The interlock occurs automatically. Make sure that the interlock is successful on both sides, on this occasion, the locking pin (1) must be visible. In order to unlock the armrests move the locking button to the back (2) and swing the armrest downwards.


**ATTENTION:** It may be possible that the armrest is a little bit bent during transport, this can be post bent easily until the locking button in front properly locks.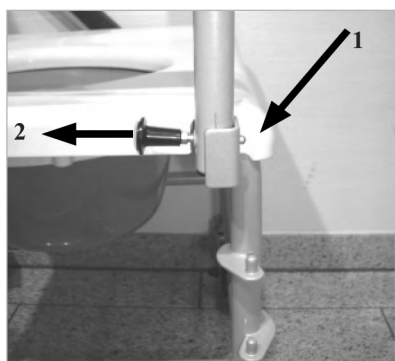


## DOUBLE FUNCTION FULL STOP BRAKE

To interlock the brake press lever downwards. This will lock the direction of the wheel in place. To open brake press other lever.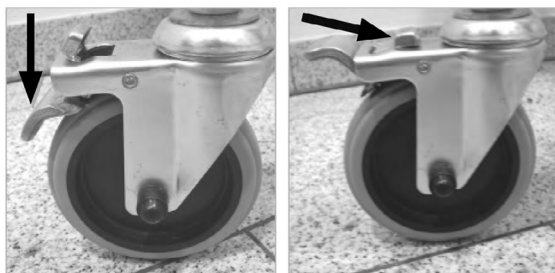

## FITTING FOOTRESTS

Hang up the provided footrests with both drillings as shown in the respective hold pins.

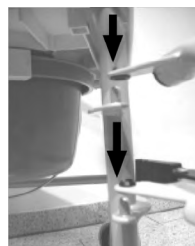## INTERLOCK AND UNLOCK FOOTRESTS

In order to interlock the footrests turn them to the middle until its audible engage. In order to unlock press the lever as shown inwards.

## HEIGHT ADJUSTMENT OF FOOTRESTS

To adjust the footrests in the height first unlock the fixing clamp (1). After pinching the fixing bolts (2) the footrests can be shifted (3) in 4 steps in the height for leg lengths from 420 – 500 mm.

**ATTENTION:** Never use the foot rest as stand up device!

## BUCKET

Insert or remove the toilet bucket (also with put on lid) always from the back. The toilet bucket engages, the toilet seat can be taken simply upwards for cleaning.

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**SUNGO EUROPE B.V.**