Aquatec® Step

en	Stepping aid	1
	User Manual	
de	Einstiegshilfe	3
	Gebrauchsanweisung	
fr	Marchepied	5
	Manuel d'utilisation	
it	Ausilio per salita	6
	Manuale d'uso	
es	Escalón de ayuda	8
	Manual del usuario	
pt	Dispositivo de escalada	10
	Manual de utilização	
nl	Opstapje	12
	Gebruiksaanwijzing	
lt	Įlipimo pagalba	13
	Naudojimosi instrukcija	

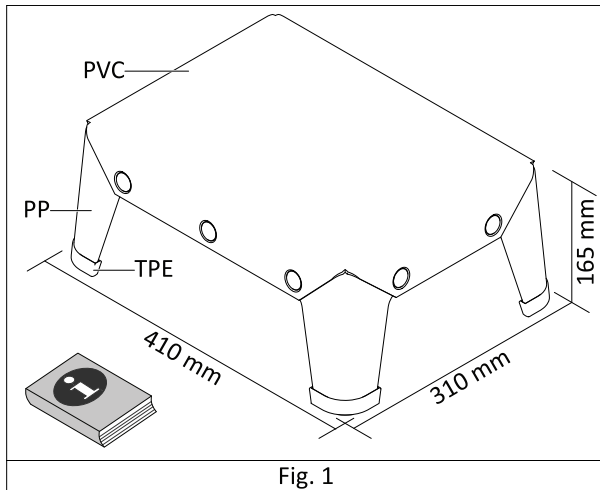
Fig. 1

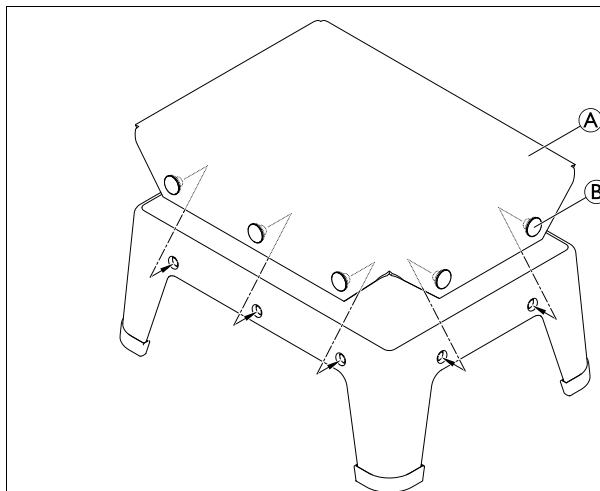
Fig. 2

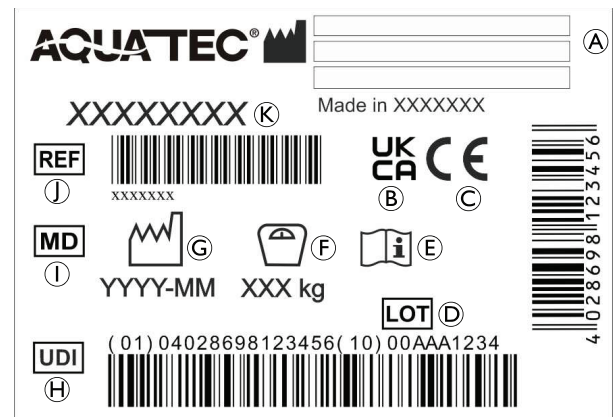
Fig. 3

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions. If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website.

Invacare reserves the right to alter product specifications without further notice.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the product was obtained.

1.3 Compliance

This CE- and UKCA-marked medical device conforms to: EU Medical Device Regulation, UK Medical Device Regulation, ISO 13485, ISO 14971, REACH, EN 12182 and related standards.

AQUATEC®

This manual **MUST** be given to the user of the product.
BEFORE using this product, this manual **MUST** be read and saved for future reference.

Yes, you can.®

1.4 Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material.

WARNING!

Risk of Falling!

- Before every use, check the product for damage and ensure that it is securely assembled.
- Do not use if defective.
- Do not make any unauthorised alterations or modifications to the product.
- Do not move or displace the product under load or during use.
- Only use the product on level ground.
- Only use accessories and options intended for this product.

2.2 Labels and Symbols on the Product (Fig. 3)

(A)	Manufacturer	(G)	Production date
(B)	UK Conformity Assessed	(H)	Unique Device Identifier
(C)	European Conformity	(I)	Medical Device
(D)	Lot number	(J)	Reference number
(E)	Read user manual	(K)	Product designation
(F)	Maximum user weight		

The identification label is attached to the underside of the product.

3 Product Overview

3.1 Intended use

This stepping aid is intended as an aid to enter into or get out of a bathtub for persons with mobility limitations. Indication: If used without an assistant, the users must be able to keep their own balance. There are no

contraindications known when using the product as intended. For the maximum user weight, see 7 *Technical Data*, page 3.

3.2 Scope of Delivery

The scope of delivery is shown in figure 1.

4 Setup

For removing or installing the non-slip cover (A), refer to figure 2.

WARNING!

Risk of Slipping

- Make sure that all 10 fastening buttons (B) are securely fastened before using the product.

5 Maintenance

5.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

5.2 Cleaning and Disinfection

General Safety Information

CAUTION!

Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

NOTICE!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

Cleaning Intervals

NOTICE!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product:

- before first use,
- regularly while it is in use,
- when it has been in contact with any body fluids.

Cleaning

1. Remove detachable parts.
2. Clean all parts by wiping down all generally accessible surfaces.
3. Machine-wash the cover at max. 60 °C.
4. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

6 After Use

6.1 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *5 Maintenance, page 2*.

Make sure that the user manual is handed over with the product.

6.2 Disposal

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

7 Technical Data

7.1 Weights

Product weight: 1.5 kg Max. user weight: 150 kg

All product components do not contain natural rubber latex.

7.2 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare AG
Invacare New Zealand Ltd
Invacare GmbH
Invacare UK Operations Limited