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## INSTRUCTIONS MANUAL EN

## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a assistive device. He or she will advise you on the most appropriate use for you.

Distributor: This manual must be given to users of this product.

## PRODUCT DESTINATION

The Slide access chair is a medical device for people with reduced mobility. It allows you to sit comfortably out of your bathtub and slide into it without getting up. It is equipped with a space to place your shower

products within easy reach. Can be used on most standard bathtubs. Stay seated and slide into the centre of your bath thanks to our Slide bath access chair.

## PRODUCT DESCRIPTION

## SAFETY PRECAUTIONS

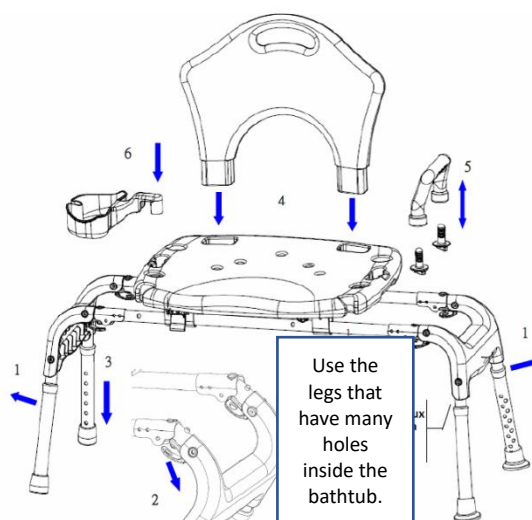
Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum authorized limit of 130 kg
- Make sure the product is in good working order
- Check that all parts are properly engaged
- Make sure that the user locks the latch allowing the transfer of the seat
- In case of damaged, broken, worn or missing parts, contact the dealer.
- Make sure that the bar that holds the 4 feet is horizontal
- Make sure that the studs of the 4 feet come out of the 4 feet
- Beware of the risk of pinching when using


## MOUNTING

1. Fold the shower chair out and insert the feet until you hear the click.
2. Clip the assembly as shown in 2.
3. Adjust the feet to the desired height and adapted to the user.
4. Insert the backrest by pressing down strongly.
5. Attach the handle as shown in 5
6. Position the soap dish as shown in 6

## USE

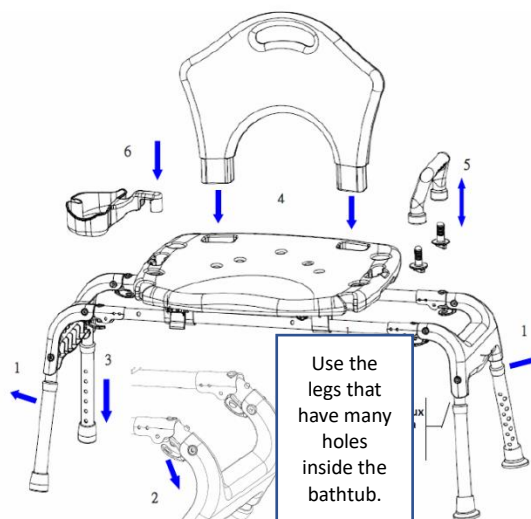1. Pull on the red handle located under the seat to slide the seat laterally on the frame to the position
2. For maintenance, use soapy water. Rinse well before use.

## CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.
- Checklist :

To keep the product in good condition, check before each use, and check regularly as follows:

Check the article	Anytime	Weekly
Check that the chair slides well along the structure	X	
Check the condition of the seat		X
Check the operation of the sliding blocking system	X	
Check the operation of the 4 height adjustment clips	X	

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Respect the conditions for storing and storing your product.

- In a dry and temperate place
- Protect your product by packaging from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the components of the product, do not jam anything...).

## PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

## FEATURES TECHNICAL

Overall dimensions	width 106 x depth 46.5 x height 79.5/89.5 cm.
Seat dimensions	width 57 x depth 47 x height. 54/66 cm.
Material	Structure aluminium anti-corrosion.
Weight	6.5 kg
Maximum weight authorized	130 kg

## GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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