## CHAISE CONFORT HAUTE CAPITONNEE

REF : 812057 - 812057.GRIS

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49124 Saint Barthélemy d'ANJOU

## PERCHING STOOL – REF : 812057 – 812057.GRIS


### Installation instructions and user guide

Thank you for choosing this Perching stool with PU seat and steel back and armrests. This user guide should be read carefully before use as it contains important information on the safe operation and maintenance of the products.

Product details and dimensions	
Width	560-635 mm
Depth	450-460 mm
Seat width	325 mm
Seat depth	265 mm
Seat height	535-685 mm
Width between arms	480 mm
Height of armrests from seat	175 mm
Maximum user weight	160 kg


**WARNING: maximum user weight: 160 kg**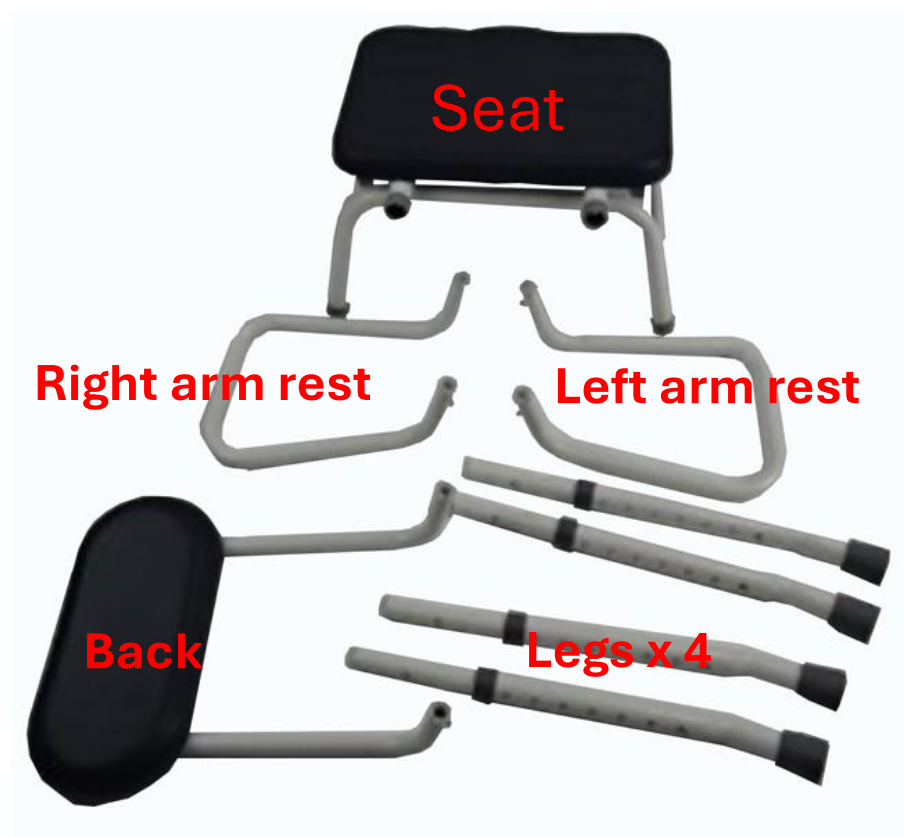

Check that all the parts are present. No tools are required for assembly.

Please follow instructions 1 – 7

1

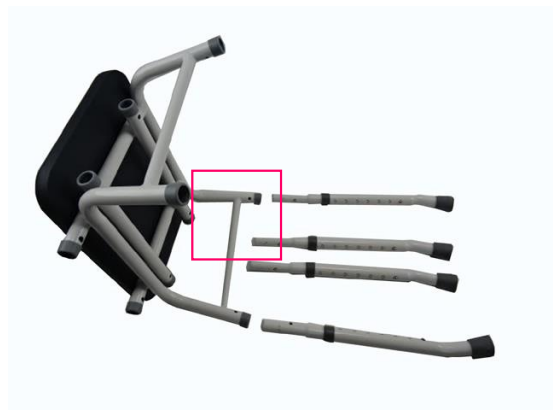Fit the legs first

2

Insert the legs into the sockets in the seat base. Align the small silver button, press the button and push the leg into the socket.

3

When the leg is correctly positioned, the button will appear in the hole and lock it in place.

4

Mount the other three legs and stand the stool upright

5

Then mount the armrests.

The armrests have a particular shape.

The red arrow points to the front

The blue arrow points to the back of the seat, identified by the backrest sockets.

6

Fit the second armrest and while pressing the buttons, slide the armrests until they lock into place.

7

Mount the backrest in the same way.

## Using the perching stool

- When moving, always lift the perching stool by placing your hands on the tubular metal frame under the seat.
- Do not allow children or unauthorized persons to play with or use the equipment without proper supervision.
- Do not rock back and forth or side to side when using the stool as this will cause excessive strain and possible damage and failure which could result in injury.
- Do not use the perch stool outside the house or on uneven surfaces (such as in the garden) as this will cause strain on the legs and possible damage leading to injury.
- Do not place fingers or foreign objects in any of the exposed tubular sections as this may result in serious injury.

## Cleaning

Before cleaning the chair, please remove all the pin clips and the four extension feet with the end caps fitted. Clean these components separately from the main frame.

Clean the upholstery using a soft, non-abrasive cloth and a mild detergent. Abrasive cleaners and/or abrasive cleaning pads will seriously damage the upholstery beyond repair.

## Instructions for use

To adjust the height, press the silver button on each leg, move the leg to the desired height, ensuring that the button is securely fixed in a hole before moving on to the next leg. Extend all the legs evenly to the desired height position, taking care not to go beyond the last drilled hole in the extension leg.

Note that the perching stool is designed to tilt forward. Do not adjust any of the legs so that the seat is level, as this will put excessive strain on the legs, compromising the user's safety and potentially causing permanent damage to the perching stool, including fracture of tubular and other sections.

## Check

Periodically check that all components are present, correctly assembled and correctly tightened.

### **Distributed by :**

Identites – ZA Pole 49

Bd de la Chanterie

49124 Saint Barthélemy d'ANJOU