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N° de série :

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Works with an inoffensive 9 V pocket battery (not included).

The aim of this machine is to cure the most frequent cause of Enuresis : the deep of sleep ; the child has a too deep sleep and doesn't wake up to urinate. Awakening the child at fixed times is inefficient. It's essential to wake the heavy sleeper child up, as soon as he begins urinating. The most easy and normal method is the sound excitation induced by the machine «Eduque-Reflex PIPI-STOP». The ringing is activated after the first drops of urine, waking then the child up. The child must be completely woken and taken to the toilet in order to finish urinating. After a few days (exceptionally a few weeks) the child wakes by himself at times which come quickly close to the morning. It's necessary to simply explain the child the cause of his enuresis (deep sleep). Then suggest the help of a ringing at the night time. You have to insist on the fact it's not a punishment but the best way of curing him quickly.

INSTRUCTIONS FOR USE :

Put the strip at the bottom of underpants, put the absorbent side against the skin in front of the urinary orifice. You must use a perfectly dry strip. It's important to rinse abundantly the strip with hot water after use (NOT TO WRING THE STRIP). To stop the ringing, disconnect the strip of the machine.

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